



# Rapid White Travel Tooth Whitening Pen

- Fast, easy and effective whitening
- Perfect "on the go" size for handbag or pocket
- Clinically proven

The Rapid White Travel Tooth Whitening Pen is a gentle treatment which whitens teeth up to **4 shades whiter** in 7 days by targeting and removing stains, helping you to get a whiter, brighter, younger-looking smile without using peroxide.

Easy to use, safe and convenient, this pen is designed to fit into your handbag or pocket, for use at home or on the move.

## Instructions: FOR ADULT USE ONLY

For best results gently brush teeth before use.

**Step 1:** Position the pen vertically so that the Accelerator is at the bottom end. Twist the centre barrel to open, carefully remove the brush and apply a coat of the Accelerator to each tooth to be treated. Replace cap securely.

**Step 2:** Immediately after application, turn the pen over and twist the centre barrel to open the Whitening Gel. Again, carefully brush a coating onto each tooth to be treated. Do not overcoat the teeth with Gel as this will not improve the whitening effect. Replace cap securely. During application, and for 30 seconds afterwards, keep your lips and tongue away from your teeth.

### For best results:

Do not eat or drink for 30 minutes after application.  
Use twice a day for 7 days. Continue to use for additional whitening for up to 14 days, or for touch ups as required. Use the enclosed Professional Tooth Shade Guide to monitor your progress.

### Pack contents:

Tooth Whitening Pen with dual compartment



Before & after



CAUTIONARY STATEMENT: This product is not recommended for use by anyone under 14 years of age or individuals with gum disease, gingivitis, bleeding or severely receding gums, tooth sensitivity, pregnant or nursing mothers. As with any product a review of the ingredients is recommended to help avoid any allergic reaction. This product works on natural human teeth only, it will not harm fillings, crowns and caps. Avoid contact with lips and gums. Do not use while wearing dental braces. Use for periods of longer than 14 days should only be under the advice and supervision of a dentist. If irritation occurs, discontinue use immediately.

SENSITIVE ACTION FORMULA



IT'S TIME TO  
**SMILE**